


Instructions



Radiological control
Push the caliber toward the root and read at the extremity the amount to remove.



Once the dam has been applied level the outer margin of the root

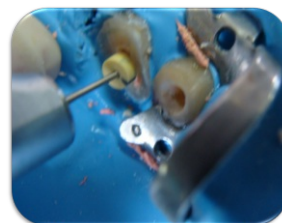

Shape the canal with the burr large up to 2/3 of the canal under radiological control



Place the guide axis inside the canal and drill until it reaches the outer margin of the root



Introduce the caliber inside the canal with the measuring pin in a "0" position



Push the caliber towards the root and read at the extremity the amount to remove.



Insert the pin on the ruler, mark and cut the excessive part



Carry Put all the procedures for the cementing and remove the applicator



Etching Gel

Instructions



Etching the root canal




Dual Band



Insert the pin in the root



Dual cement

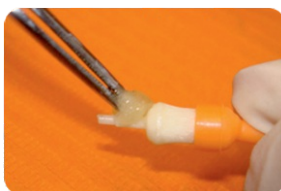


Apply the dual band in the pin, cure and apply the cement in the pin



Apply the band in the root canal



Remove the applicator

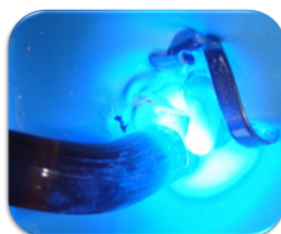

Cure with halogen lamp for composites



Complete the preparation of the pin by diamond burr